Applic - Photopolimerizable Nanohy Compound Resin

ENGLISH

Content:

01 syringe with 4 grams in colors EA1 / EA2 / EA3 / EA3,5 / EA4 / EB1 / EB2 / EB3 / EC2 / EC3 DA1 / DA2 / DA3 / DA3,5 / DB2 / INCISAL.

Method of Use:

1. Begin Prophylaxis.
2. Select color with teeth clean and moisturized. The color can be selected using the Vita Classical scale or with the product itself, placing a small portion of it on the tooth to be treated, and without acid conditioning and adhesive system, polymerize for 30 seconds.
3. Isolate area absolutely.
4. Carry out the acid conditioning (AcidGel - Maquira) of the enamel for 30 seconds and dentin for 15 seconds. Then wash the area with plenty of water. Dry with light air jets avoiding dehydration of the dentin.
5. Apply the adhesive according to the manufacturer's instructions.
6. Place the PHOTOPOLIMIZABLE NANOHYBRID COMPOSITE RESIN APPLICATION, according to the selected color, in layers from 1mm to 2mm, photopolymerizing according to the photopolymerization times in the table below:

Colors	Photopolymerization times	Colors	Photopolymerization times
Ec1/ EA2/ EA3 / EA3,5/ EA4 / EB1/ EB2 / EB3 / INCISAL	20 seconds	EC2 / DA1 / DA2 / DA3 / DA3,5 / DB2	40 seconds

7. Finish and polish with sandpaper discs and felt discs, using polishing pastes (Diamond Universal - Maquira).
8. Check the occlusion.

Precautions:

Exclusive use for dental surgeon.

Protect the pulp with a suitable base before beginning the acid conditioning.

Products such as eugenol or clove oil inhibit polymerization.

Close the syringe immediately after use.

The use of cationic solutions, plaque developers and chlorhexidine may discolor the resin.

Do not use the product passed the expiration date.

Keep out of reach of children

Wear safety goggles when handling the photopolymerizer.

Transportation and Storage Conditions:

Store the product in a dry, well-ventilated place away from direct sunlight. Do not store near materials that contain eugenol or clove oil.

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