Hemostatic solution for topical use.**Presentation:** 1 Bottle with 10 mL.

Description: Hemopare helps gingival retraction and hemostasis, due to the presence of aluminum chloride, which causes a contraction of the superficial layer of the free gingiva by the precipitation of serum and tissue proteins. It is used in cavities for restorations and dental prostheses. The conditioned tissues restore within 1 hour. **DO NOT CONTAIN EPINEPHRINE.**

Indication: Hemostasy and gingival retraction.**Composition:** Aluminum chloride, sulfate of hydroxyquinoline and ethyl alcohol.**How to use:**

- With the aid of a cotton swab dipped in the solution compress the place to be conditioned until it stops the bleeding.
- For cervical matrix application, use Hemopare solution in the cervical area before its placement, preventing contamination.
- Let the product act for 5 minutes and rinse it with water/air jet.

Warnings and precautions:

- Restricted use, for professionals who specialize in dentistry, in accordance with local legislation;
- Rigorously follow the procedures provided in the Use Instructions;
- Do not let product come into contact with eyes and skin;
- Do not use the product after it has passed its expiration date.
- Keep out of reach of children.

Contraindications and collateral effects: It should not be used in people with reported sensitivity to its components.**Storage and transport:** Always store the product in its original packaging, at room temperature. Protect it from direct exposure to sunlight.**Method of discard:** Discard the product in accordance with local legislation, assuring its complete disfigurement in order to prevent its reuse and damage to the environment.**Validity:** 3 years.

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