**PRESENTATION:**

1 unit - Autoclavable Flex Retractor size L / M;

1 unit - Expandex Lip Retractor Adult / Children size Autoclavable;

2 units - Lateral Cheek in U Adult / Child size Autoclavable;

2 units - Lateral Cheek in V Adult / Child size Autoclavable.

**DESCRIPTION:**

The retractor is a medical device that assists in the opening of the lips, facilitating various procedures in dentistry. Its use is basically indicated to remove the labial mucosa and cheek for brackets bonding, prophylaxis, intraoral photographs, fluoride application and adhesive restorations, providing a greater opening of lips and cheeks. It features modern, comfortable and safe design for patient use.

**INDICATION:**

Assisting intraoral photographs of posterior teeth, brackets bonding, prophylaxis, fluoride application and adhesive restorations.

**COMPOSITION:**

Expandex and Flex: Polyacetal. U and V-shaped side: Polycarbonate.

**CLEANING AND STERILIZATION PROCEDURE:**

1. After using the product, make the removal of organic matter in ultrasonic tubs by using enzymatic soap (minimum 3 enzymes) under dilution recommended by the manufacturer.

2. Rinse under running water and observe the presence of residues, repeat the process or make manual removal.

3. For autoclave sterilization, after cleaning, arrange the parts in the packaging for sterilization and autoclave at 121 °C temperature (250 °F), according to the autoclave manufacturer's recommendations.

Observation: Retractors have a validity of 50 cycles for sterilization.

**MODE OF USE:**

1. First, make sure the product is clean and sterilized;
2. Moisten the lips of the patient;
3. Position the instrument as necessary skills and training;
4. After using perform the cleaning and sterilization process. Store the product in the package itself for sterilization.

**WARNINGS AND PRECAUTIONS:**

The use is restricted to professionals specialized in dentistry, according to local legislation and with specific training for its handling. Procedures provided in the directions of use should be strictly followed. Use only after the application of cleaning and sterilization procedures. Use the correct size of retractor for each patient. Do not use the product after the expiration of the allowed cycles. Keep away from the reach of children.

**CONTRAINDICATIONS AND SIDE EFFECTS:**

Patients with sensitivity to polycarbonate or polyacetal should not use the product.

**STORAGE AND TRANSPORT:**

Store in the sterilization packaging, at room temperature, away from heat, direct sunlight and humidity. Temperature: less than 50 °C. Humidity: below 75 UR.

**DISPOSAL:**

The product must be disposed of according to the legislation in force in your country.

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