PRESENTATION:

Adult/Child Expandex Retractor – 1 unit.

DESCRIPTION:

The Maquira Expandex Chemical Retractor was developed to assist the professional in various procedures in dentistry. It provides a greater opening of the lips and cheeks without the need to use the professional or assistant's hands. It has a modern design, comfortable and safe for use in the patient.

INDICATION:

Indicated to assist in procedures that require removal of lips and cheeks for greater dental exposure, including: bracket bonding, prophylaxis, fluoride application, adhesive restorations, dental whitening in the office, application of desensitizers, prosthetic preparations and frontal intraoral photographs.

COMPOSITION:

Polycarbonate.

CLEANING AND DISINFECTION PROCEDURE:

1. After each use of the product, remove the organic matter in ultrasonic tanks, using enzymatic soap (minimum 3 enzymes) at the dilution recommended by the manufacturer; 2. Rinse in running water and observe the presence of residues, repeat the process or perform manual removal; 3. For the disinfection procedure, place the product in 0.2% peracetic acid solution for 60 minutes. Rinse well under running water and store in a closed container.

MODE OF USE:

1. Check if the product has passed through the cleaning and disinfection procedures; 2. Wet the patient's lips; 3. Reduce the size of the retractor by activating it by the fins of the shells, approaching them; 4. Insert the retractor starting from one side of the

mouth, keeping the shells pressed; 5. Position the instrument and fit the upper and lower lips according to the necessary skill and training; 6. Ask the patient to close their mouth after accommodation of the retractor; 7. After its use, carry out the cleaning and disinfection process. Store in a closed container.

WARNINGS AND PRECAUTIONS:

Restricted use by specialized dental professionals, according to local legislation and with specific training for their handling. Strictly follow the procedures provided in the mode of use. Use only after cleaning and disinfection procedures. Use the correct retractor size for each patient. Check the condition of this product before each use and purchase a new one if excessive fatigue or risk of breakage is observed. Keep out of reach of children.

CONTRAINDICATIONS:

Do not use in patients with sensitivity to polycarbonate.

STORAGE AND TRANSPORT:

Store the packaging in a clean place and away from direct sunlight. Temperature between 3 °C to 50 °C.

DISPOSAL:

The product must be disposed of in accordance with the current legislation of your country.

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