Presentation:

2 syringes with 3 g and 6 tips.

Indication:

Indicated for disinfection of cavity preparations for crows and inlays.

Description of the Product:

The CLORHEXIDINE GEL 2% is an antiseptic with antibacterial activity with effectiveness for Gram-positive and gram-negative (including *Pseudomonas*), fungi, yeasts and some dermatophytes. Its viscosity allows the product does not run during application, remaining inside the dental cavity.

Composition:

Clorhexidine digluconate 2%, methylparaben, propyleneglycol, hydroxyethyl cellulose and purified water.

How to use:

1. Fill the entire canal with the gel 2% and instrument according to the need;
2. Remove the excess and dry the cavity with a jet of air.

Precautions:

1. Avoid the ingestion of the product;
2. Avoid the contact of the product with the tongue and mouth regions. May cause staining at the teeth, tongue, and piercings and transient change of taste;
3. Do not use the tips to apply the product directly to the patient. Place the material into another container and make the application with sterilized instrument;
4. Do not use the product with the expiry date;
5. Do not use in patients with known sensitivity to Clorhexidine.

Storage and Transportation Conditions:

Store in a clean, dry place, protected from direct sunlight.

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