Content:

2 Syringes with 3 g and 6 tips.

Description:

EDTA is generally accepted as the most effective chelating agent. When it is placed inside the pulp cavity, it acts on the dentinal walls, demineralizing them and making them less resistant to the action of endodontic instruments, which facilitates the preparation of the canal. It also removes effectively the smear layer, increasing the dentin permeability.

Indication:

EDTA is used to enlarge root canals, remove the smear layer and prepare the dentin wall for the adhesion of filling materials.

Composition:

EDTA, sodium hydroxide, thickener and purified water.

How to Use:

1. Place small amount of trisodium EDTA gel with the aid of the plastic tip at the entrance to the start of the root canal instrumentation;
2. Wait 2-3 minutes for the product starts to work and so do the instrumentation;
3. After 5 to 7 minutes the product will diminish its effect, because it reacted with dentin and neutered. Keep the canal wetted with the product during the surgery;
4. At the end of the procedure, flush the canal with an irrigating solution of sodium hypochlorite.

Precautions:

For professional use only.

Keep out of reach of children.

Do not use the product if it is expired.

Do not ingest, if it occurs seek medical advice.

Storage and Transportation Conditions:

Store the package in a clean place, away from direct sunlight, at a temperature up to 30 °C and humidity up to 75 UR.

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MADE IN BRAZIL

maquira[®]
Dental Group**Fabricante | Manufacturer:**

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