Content:

Bottle with 20 mL.

Product description:

EDTA is generally accepted as the most effective chelating agent. When it is placed inside the pulp cavity, it acts on the dentinal walls, demineralizing them and making them less resistant to the action of endodontic instruments, which facilitates the preparation of the canal. It also removes effectively the smear layer, increasing the dentin permeability.

Indication:

EDTA is used to enlarge root canals, remove smear layer and prepare the dentin wall for filling materials.

Composition:

17% disodium EDTA, sodium hydroxide, purified water.

How to Use:

1. For an effective removal of organic and inorganic components of the smear layer, the canal must be irrigated with 1 or 2 drops of 17% TRISODIUM EDTA.
2. Wait 2-3 minutes to start instrumentation.
3. After 5-6 minutes the product decreases its effect as already reacted with the dentin and are neutralized.
4. Irrigate the canal again and repeat the process according to need.
5. Keep the canal moistened with the product during the whole procedure.
6. Finishing the instrumentation, irrigate the canal using an irrigating solution, such as sodium hypochlorite.

Precautions:

For professional use only.
Keep out of reach of children.
Do not use the product if it is expired.
Do not ingest, if it occurs seek medical advice.

Storage and Transportation Conditions:

Store the package in a clean place, away from direct sunlight, at a temperature up to 30 °C and humidity up to 75 UR.

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MADE IN BRAZIL

maquira[®]
Dental Group**Fabricante | Manufacturer:**

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