Content: Bottle with 10 mL.

Description of the product: Eucalyptol is a colorless oily or light-yellow liquid with a faint characteristic aroma of camphor and a pungent and refreshing taste of camphor. It is completely insoluble in water. It is readily soluble in ethanol at 70%. It is miscible in absolute alcohol and fatty oils.

Indication: Eucalyptol is indicated as a solvent of gutta-percha cones, assisting in the removal of obturators from the root canal.

Composition: Eucalyptol.

Precautions:

● Surgeon-dentist use only. ● The product is manufactured to be used only according to the instructions for use. Separate use is the sole and exclusive responsibility of the professional who uses it. ● Use PPE when handling the product, including protective gloves and goggles. It is recommended that patients also wear protective eyewear. ● It is recommended that absolute isolation from the operating field be used when handling this product. ● Avoid contact of the product with eyes and skin. ● Do not ingest. ● In case of allergic reaction, stop using immediately. If necessary, seek medical advice. ● Do not use the product out of date. ● Keep out of reach of children. ● Single-use product. ● Dispose off after use. ● Reprocessing prohibited. ● Nonsterile.

How to use: 1. Initially perform the absolute isolation of the operative field in order to protect the patient from contact and absorption of the product by the mucosa. 2. Apply a drop of Eucalyptol at the entrance of the root canals and wait approximately 5 minutes for the product to soften the gutta-percha cone. 3. Remove the gutta-percha cone little by little, with the help of an endodontic file. 4. Reapply the product and repeat the process as many times as necessary. 5. For best results use the pure Eucalyptol.

Storage and Transportation Conditions: Always store the product in a clean, dry place, in its original closed packaging to avoid volatilization of the product, under temperature below 30 °C. Protect from direct sunlight.

Contraindications: Do not use in patients with hypersensitivity to the components of the product.

Validity: The expiration date on the packaging represents the maximum period for use of the product. Do not use after the expiration date.

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