Content:

Bottle with 20 mL.

Description of the product:

Eugenol is a colorless or yellow oily liquid of strong carnation aroma and spicy flavor. The zinc oxide and eugenol pastes have good biological compatibility, which might influence the repair of the pulp organ as well as the surgical wounds. It is a little soluble in water and completely soluble in alcohol, ether, fixed oils, and glacial acetic acid.

Indication:

Eugenol is used in the composition of several dentistry products, such as in zinc oxide-eugenol pastes, with the formation of cement sealer and in temporization cements, with analgesic and antiseptic functions.

Composition:

Eugenol.

How to use:

Add eugenol in appropriate proportions to the zinc oxide until it becomes a paste. The process requires vigorous and prolonged spatulation. A relationship powder/liquid of 3:1 or 4:1 must be used to obtain maximum resistance.

Precautions:

Professional use only.
Avoid the contact with eyes, mucosae and skin.
Keep the product out of reach of children.
Do not use the product with expired shelf life.
Do not ingest. If this happens, seek a doctor aid.
Not sterile.

Contraindication:

It is contraindicated in the cases of biopulpectomy, for it may cause pulp necrosis.
Do not use it if allergy is shown to the product.

Interactions:

Do not use in combination with resins and glass ionomers, for it might inhibit the curing and the adhesion of these materials.

Conditions for storage and transportation:

Store tightly closed in a clean, dry place, protected from light at a temperature below 30 °C and humidity less than 75 UR.

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MADE IN BRAZIL

maquira[®]
Dental Group**Fabricante | Manufacturer:**

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