**PRESENTATION:**

1 Syringe with 2 g and 5 tips.

**DESCRIPTION:**

FIX COMFORT is a photopolymerizable, thixotropic, easy to apply resin, which generates a physical barrier over the components of protuberant or fixed appliances which may cause injury to the soft tissues of the mouth, providing an additional comfort to patients. Since it is a translucent product, it does not interfere with the aesthetics of the appliance. The low viscosity of the product enables its application and increases its capacity of filling. Due to its mechanical features, the product may be easily withdrawn from the surfaces of the brackets if necessary.

**INDICATION:**

FIX COMFORT is used to protect and partially coat components of fixed appliances (brackets, hooks, mini implants) which may cause injury to the soft tissues of the mouth, creating a rounded, comfortable coat to the patient.

**COMPOSITION:**

BisGMA, HEMA, Photoinitiators, thickeners, and preservatives.

**MODE OF USE:**

1. Identify the components of the fixed appliance which are causing the discomfort; 2. Put the lip retractor to retract the cheek and the lips, dry the components which will be coated prior to the application of Fix Comfort; 3. Put the needle in the syringe and apply the product directly on the intended area. If necessary, make an adjustment with an instrument before the photopolymerization; 4. Light cure for 30 seconds; 5. Confirm with the patient if the protection was effective;

6. Its removal is easier and safer when using instruments.

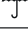
**WARNINGS AND PRECAUTIONS:**

- Exclusive use of the professional.
- Do not apply the product to components which interfere on the mechanics of the treatment, such as the wire and the ligature between wire/bracket.
- Close the syringe immediately after use.
- Use goggles while handling the photopolymerizer.
- When photopolymerizing, the use of a photopolymerizer with a power above 400 mW/cm<sup>2</sup> is recommended.
- Products such as eugenol or clove oil inhibit the polymerization.
- In case of accidental contact with the eyes, rinse the area with water and call a doctor immediately.
- Do not use the product on patients allergic to acrylates/methacrylates.
- Do not use the product if it is expired.
- Keep the product away from children.

**STORAGE AND TRANSPORT:**

Store the syringe in a fresh, dry place, protected from light. Keep in a temperature range from 5 °C to 35 °C. Do not keep near materials which contain eugenol or clove oil in its composition.

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