FORMOCRESOL

Fabricante | Manufacturer:
Maquira Indústria de Produtos Odontológicos S.A.
CNPJ: 05.823.205/0001-90
Endereço | Dirección | Address:
Av. Melvin Jones, 773 | Pq. Ind. Bandeirantes
Maringá - PR | 87070-030 | Brasil
Resp. Técnico | Technical Resp.:
Cristiane Furlan | CRF-PR: 14.038
Reg. ANVISA N°: 80322400017

SAP:
Atendimento ao cliente
Atención al cliente
Customer service:
0800 726 5848
sap@maquira.com.br

Simbologia | Simbologia | Symbology

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VALIDEZ:
24 meses.

CONTENT:
Bottle with 10 mL.

INDICATION:
Formocresol is indicated for root therapy of deciduous teeth and in some particular cases of permanent teeth.

DESCRIPTION:
This solution is formed by a mixture of cresols and formol, being a vital pulp holder whose function is to keep them inert and conserve the deciduous tooth until its physiological fall (pulpotomy). It is a powerful antimicrobial agent that has the ability to neutralize toxins in the root canal.

COMPOSITION:
Formaldehyde, O-cresol, glycerin and purified water.

HOW TO USE:
1. Anesthetize and make tooth isolation with rubber dam; 2. Remove all the carious tissue; 3. Remove the roof of the pulp chamber, taking care to avoid perforation in the bifurcation of the molars; 4. Make use of pulpotomy procedures (removal of coronal pulp); 5. Make the hemostasis and washing of the pulp chamber with detergent and humectant; 6. Apply a cotton ball soaked with formocresol (without excess) on the pulp stumps for 5 to 7 minutes to promote hemostasis; 7. Remove the cotton and put a layer of zinc oxide and eugenol and another one with zinc phosphate; 8. Restore with amalgam or steel crown.

PRECAUTIONS:
Product for professional use only.
Product contains toxic substances.
Avoid contact with gingival tissues
Keep out of reach of children.
Do not swallow it. If this occurs, seek medical help.
Not sterile.

STORAGE AND TRANSPORTATION CONDITIONS:
Store tightly closed in a clean, dry and dark local.

VALIDITY:
24 months.