**PRESENTATION:**

1 Bottle longlass R powder with 10 g, 1 bottle longlass R liquid with 8 mL, 1 measure spoon and 1 spatulation block.

**DESCRIPTION:**

longlass R is a highly resistant, aesthetic, self-curing glass ionomer with adequate viscosity and ideal for restorative work. It has excellent biocompatibility and a low risk of post-operative sensitivity. It has an optimal marginal sealing in the cavity and inhibits gap formation.

**INDICATION OF USE:**

IONGLASS R is indicated for ART (Atraumatic Restorative Treatment) technique, in all classes of restorations, provisional restorations in permanent teeth (classes I, II, III and IV), base of definitive restoration, non-carious lesions and provisional cementation of prosthetic pieces.

**COMPOSITION:**

Powder: Glass Powder.  
Liquid: Polyacrylic acid.

**MODE OF USE:**

- The operative field must be kept free of contamination. Use effective absolute isolation or relative isolation.
- Remove decayed tissue, prepare the cavity, clean and dry thoroughly without retentions. In very deep cavities it is recommended to use a calcium hydroxide based liner.
- Ensure that there are no residues of other products that could have a negative effect on the properties of the material.
- Gently shake the powder before use to homogenize the product. For accurate liquid dosing, expel air before dispensing the product and turn the bottle upside down.
- Prepare the material in the following proportion: 1 shallow measure of the powder dosing spoon to 1 drop of liquid. Using the mixing pad or a glass plate, place the powder and liquid side by side. Slowly add the powder to the liquid with the aid of the plastic spatula for glass ionomer Ionomix (Maquira) until a homogeneous mixture is obtained. Take care that the mixing time does not exceed 1 minute.
- Apply the material into the cavity while still shiny, using a Precision pistol (Maquira) or a suitable instrument, taking care not to incorporate air bubbles.
- During the setting phase, avoid contact with saliva and water, because the material is vulnerable.
- Excess material can be removed with an explorer probe during the plastic phase of the product.
- The ideal working time is a maximum of 3 minutes. The setting time is a maximum of 6 minutes. After this time, excesses can be removed with suitable instruments.
- Note: The temperature range for the given times is 18 °C to 32 °C. Temperatures above decrease the working time and temperatures below increase the set times.
- Cover the restoration with a surface protection agent for protection against moisture.
- For improved finishing and polishing, wait 24 hours after insertion of the material.
- After use rinse used materials with cold water before material residue solidifies.

**WARNINGS:**

- Exclusive use of the dental surgeon.
- The adequate proportion of phases (powder and liquid) must be respected.
- Do not mix powder or liquid components of this product with any other glass ionomer based material.
- This material when in contact with eugenol, phenol or paramonochlorophenol decreases the adhesive strength, therefore it should not be used in association with these products.
- Check for the total absence of eugenol in the area, as it interferes with the setting of the material.
- In addition to adhering to enamel, dentin and cement, this product also adheres to stainless steel, tin, or tin-oxide-coated platinum, and gold. However, it does not adhere to porcelain, pure platinum, or pure gold. Do not use in the cementation of laminated veneers and bonding of brackets.
- Do not allow the product to come into contact with the eyes, soft tissues or skin. In case of contact, wash immediately with plenty of water. In case of allergic reactions discontinue use immediately. If necessary seek medical assistance.
- Do not use the product after its expiration date.
- Single use product.
- Reprocessing forbidden.

**PRECAUTIONS:**

- Wear PPE when handling the product, including gloves and goggles. It is recommended that patients also wear goggles.
- Absolute isolation of the operative field is recommended when handling this product, as loose objects can be accidentally swallowed or aspirated.
- After use, close well the powder and liquid bottles to avoid contact with humidity.
- Keep out of reach of children.

**CONTRAINDICATIONS:**

Do not use in patients with hypersensitivity to product components.

**ADVERSE EFFECTS:**

Application of the product directly on the dental pulp may cause irritation or irreversible damage (pulpal death).

**STORAGE AND TRANSPORT CONDITIONS:**

Store product in closed package, at temperature up to 30 °C, humidity up to 75 UR, in a dry place, protected from light and heat.

**DISPOSAL:**

The product must be disposed of according to the current legislation of your country.

**VALIDITY:**

The expiration date that appears on the packaging represents the maximum period for using the product. Do not use after expiration date.

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