**PRESENTATION:**

1. 50 units of S, M and L Universal Matrix assorted, 2 units of Clamps and 8 units of Protectors.

2. 25 units of S, M and L Universal Matrix assorted, 1 unit of Clamps and 4 units of Protectors.

3. 50 units of Universal Matrix S.

4. 50 units of Universal Matrix M.

5. 50 units of Universal Matrix L.

6. 50 units of Universal Matrix S, M and L assorted.

7. 2 units of Clamps and 8 units of Protectors.

8. 20 units of Protectors.

**DESCRIPTION:**

Mechanical devices with the purpose of giving the restorative material the natural anatomy of the tooth.

**INDICATION:**

Preformed, disposable matrices indicated for class II restorations and includes sectional matrices and clamps for fixation.

**COMPOSITION:**

Stainless steel, non-toxic silicone, nickel-plated steel.

**MODE OF USE:**

1. Check the size of the sectional matrix (S, M or L) to see whether it is better to adapt the cavity to be restored. Put the matrix on the proximal surface to be restored and then on the wedge and afterwards use the fixation clamps with the silicon protectors.

**Obs.:** The use or non use of clamps and silicone protectors and the way how they are installed depends on the personal preference of each user and the clinical situation.

2. Carry out the restoration process. As soon as it is completed, remove the fixation clamps and then the wedge and finally the matrix.

**WARNINGS AND PRECAUTIONS:**

Professional use. Matrices are for single use only, reprocessing prohibited. Clamps and silicone protectors after each use must be sterilized separately in an autoclave at 121 °C for 30 minutes or at 135 °C for 15 minutes. When there is a proven allergy to any of its components, the product should not be used. It is recommended to protect the papilla by any method to avoid accidents. It is recommended that the use of this product always be accompanied by absolute isolation of the operative field. Respect the expiration date. Keep out of reach of children.

**STORAGE AND TRANSPORT:**

Keep the product away from rain and direct sunlight.

Temperature: below 60 °C.

Humidity: below 95 UR.

**DISPOSAL:**

The matrices are for single use only and after use must be disposed of in hospital waste or according to local legislation.

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