Content:

Bottle with 10mL.

Indication:

Gutta-Percha solvent.

Product description:

MAQUIRA Orange Oil is an essential oil extracted from the bark of sweet orange which is volatile and with pleasant odor. It has the ability to dissolve gutta-percha, zinc oxide cement and eugenol, with or without association. It is biocompatible, does not have deleterious effects on health, it is slightly soluble in water, but easily removed from inside of the channel by surfactant substances.

Composition:

Orange Essential Oil.

How to Use:

1. Start the process of restoration removal using the mechanical method.
2. Apply 1-2 drops of Orange Oil on the root canal and allow it to act during 5-10 minutes.
3. Introduce the tools to the apical foramen and remove all the filling material inside the channel.
4. After complete removal, use a surfactant solution to promote effective cleaning.

Precautions:

1. Product of professional use only.
2. In case of contact with the oral mucosa, a mild irritation may occur on the site. Remove with cotton and wash with water.
3. Do not use the product with the term of validity expired.
4. Keep out of reach of children.
5. Contraindicated in people sensitive to orange oil.

Conditions for Transportation and Storage:

Store hermetically sealed in a clean, dry and light-protected place.

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MADE IN BRAZIL

maquira[®]
Dental Group**Fabricante | Manufacturer:**

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