Content:

Bottle with 20 mL.

Indication:

The CAMPHORATE PARAMONOCOLOROPHENOL is an antiseptic and analgesic product used as an intracanal medication (temporization).

Product description:

The CAMPHORATE PARAMONOCOLOROPHENOL is a colorless or pale yellow liquid with a camphor aroma. The presence of Phenol and Chlorine in its composition, providing antiseptic action and presenting a broad antimicrobial spectrum. Camphor is a vehicle and also decreases the irritant action of the phenolic derivative, resulting in a low-power aggression substance on the living tissue when used the shown concentration.

Composition:

Paramonochlorophenol, camphor, ethanol and purified water.

How to use:

1. Instrument and dry the root canal.
2. Soak a cotton ball in the product and dry it with a small cotton roll.
3. Place the dry cotton ball in the pulp cavity and close tightly with appropriate cement.
4. The estimated time of use is 24 to 48 hours. Then, remove the bandage and carry out root canal filling.

Precautions:

Product for dental surgeons use only.

Keep out of reach of children.

Do not use the product with expired expiration date.

Non-sterile.

This product is manufactured for dental use and should be handled according to instructions.

Transport and Storage Conditions:

Store tightly closed in a clean, cool place away from direct sunlight at a temperature between 5 °C and 30 °C.

Validity:

The product is valid for 24 months from the date of manufacture.

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