AUTOCLAVABLE ENDO+ CONE POSITIONER

EN


INDICATION:

The AUTOCLAVABLE ENDO+ CONE POSITIONER assists in guiding the angulation of the dental x-ray machine, in periodontics, endodontics and implant dentistry procedures with or without absolute isolation, resulting in standardized and clear radiographic images.

COMPOSITION:

Thermoplastic, pigment and non-toxic silicone.

PIECES:

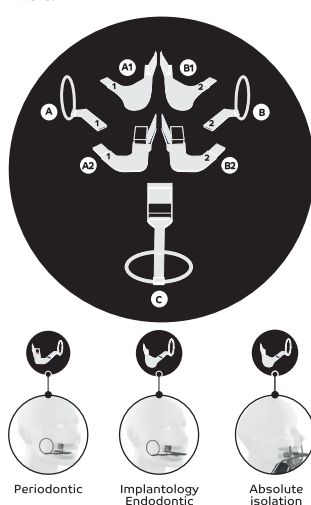ADJUSTMENT MODE:

The components of this kit are to be used as shown in the illustrations. The angle of the x-ray machine is correctly matched to the position of the film and tube parallel to the positioner in the selected procedure (periodontics, endodontics or implantology).

Periodontics: As shown in the picture, fit piece A2 to arch A to radiograph upper right lateral and lower left lateral teeth. Attach piece B2 to arch B for radiographing upper left and lower right teeth. The piece C is for radiographing upper and lower front teeth.

Endodontics – Implantology: Fit the piece A1 into the arch A to radiograph upper right lateral teeth and lower left lateral teeth. Fit the piece B1 into the arch B to radiograph upper left and lower right teeth.

MODE OF USE:

Periodontics: For positioning in the patient's mouth, the film and the tooth must be parallel to each other. During the positioning of the upper and posterior teeth the archwire must touch the forehead of the patient. In a posterior radiograph the bite is always in the area of the anterior teeth. When you want to dissociate overlapping roots, mesialize or distalize the bracket

in the mouth by plus or minus 20 degrees horizontally to achieve a distortion-free mesialization or distalization. Place the positioner in the patient's mouth so that the tooth in question is in the center of the radiographic film. When taking the X-ray film, align the cone of the X-ray machine with the ring of the positioner.

Implantology: Endo+ has a unique bite device that compensates for the overbite of the Parallelizer/Direction Indicator, and a unique shape that allows for transoperative radiographs in implant dentistry. Place the positioner in the patient's mouth so that the tooth in question is in the center of the radiographic film. On posterior teeth, the positioner is held in position by the patient, who closes his or her mouth by biting down on the bite device. On anterior teeth, the patient keeps his mouth open by holding the positioner by the stem (with thumb and index finger in the form of tweezers), dispensing with the use of the biting device. When taking the radiograph, align the x-ray machine cone to the ring of the positioner.

Endodontics: Endo+ has a unique bite device that compensates for the part of the file and cone that goes beyond the tooth crown. It can be used with or without isolation. In case of absolute isolation, loosen the rubber sheet on the opposite side of the tooth to be radiographed (picture). Place the positioner in the patient's mouth so that the tooth in question is in the center of the radiographic film. In posterior teeth, the positioner is held in position by the patient as he/she closes his/her mouth by biting down on the biting device. On anterior teeth, the patient keeps his or her mouth open by holding the positioner by the stem (with thumb and index finger in a pincer shape), dispensing with the use of the bite device. During the X-ray examination, align the x-ray machine cone to the ring of the positioner.

STORAGE:

Store the package in a clean place away from direct sunlight. Temperature between 3 °C to 50 °C and humidity up to 75 RH.

WARNINGS AND PRECAUTIONS:

Exclusive use of the professional. Do not use the product without prior cleaning and sterilization.

CLEANING/STERILIZATION:

Before starting the cleaning process, remove the biting device from the parts. Remove organic material using ultrasonic cleaners with enzymatic detergent (minimum 3 enzymes) at dilutions recommended by the detergent manufacturer. Then wash with running water. Pack the pieces in sterilization wrapping (envelopes of surgical grade paper) and put them in the autoclave with temperature from 121 °C to 134 °C, observing the recommendations of the autoclave manufacturer's instruction manual. The pot and cap are not autoclavable, they are only used to store the product after sterilization. Remove the bite device from the positioner to autoclave. Sterilization is valid for one week if the package is not violated. Chemical disinfection can also be performed using, for example, 0.2% peracetic acid solution for 1 hour. Repeat the process after each use.

EXPIRATION DATE:

After opened the positioners have an expiration of 50 sterilization cycles.

DISPOSAL MODE:

The product must be disposed of according to the legislation in force in your country.

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