DESCRIPTION:

The Precision applicator is a dental material injection system that assists in the insertion of viscous materials such as resin, cement, paste and gel in various procedures. It is able to reach areas of difficult access through its tips, quickly, accurately and without bubble formation.

INDICATION:

The Precision Applicator kit has three options of tips, each one of them is suitable for the application of products with different viscosities.

Tip nº 02 – High fluidity: It has a diameter of 0.8 mm and is mainly used in subgingival procedures, with low viscosity fluid impression materials. Its plunger has blue color.

Tip nº 03 – Heavy fluidity: It is a plastic cannula used for high viscosity materials such as thicker composites, materials of nuclei and other heavy cement. The plunger has black color.

Tip nº 04 – MEDIUM FLUIDITY: It is a curved plastic cannula recommended for composites in posterior and building materials for nuclei construction. It has 1.0 mm diameter and its plunger is green.

COMPOSITION:

Polycarbonate.

CLEANING AND STERILIZATION PROCEDURE:

Prior to use the Precision applicator, it should be washed with water and soap and then sterilized by using autoclave steam during a total cycle of 90 minutes, at a temperature of 121 °C/250 °F or according to the autoclave manufacturer specifications.

NOTE: Remove the assembly steel spring before autoclaving. This spring is not autoclavable.

* The Precision applicator has a maximum number of 50 sterilization cycles.

DIRECTIONS OF USE:

1. First make sure if the product is clean and sterilized; **2.** Manipulate the product to be used in the dental procedure, according to the manufacturer's instructions manual; **3.** Place the desired material in the specific tip, with the aid of a spatula and insert the plunger corresponding to the tip; **4.** Through the upper hole of the applicator, place the tip by pulling the plunger of the applicator backwards; **5.** Make the application; **6.** The tip along with the plunger should be discarded. Perform a cleaning and sterilization procedure.

WARNINGS AND PRECAUTIONS:

Exclusive use of professionals specialized in dentistry.

Strictly follow the procedures provided in the directions of use.

Use only after the application of cleaning and sterilization procedures.

Do not use the product after the expiration of allowed cycles.

Discard the tips with their plungers after use.

Keep away from the reach of children.

CONTRAINDICATIONS AND SIDE EFFECTS:

There are no contraindications or side effects of this product.

DISPOSAL:

Tips and plungers should be discarded according to current legislation in your country, as well as the applicator after 50 cycles of sterilization.

STORAGE CONDITIONS:

Store in a clean place at room temperature up to 35 °C, away from direct sunlight and humidity. Humidity up to 75 UR.

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maquira®

PRECISION APPLICATOR
 **MADE IN BRAZIL**

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