**INDICATIONS:**

Indicated for temporary cementation of single or multiple prosthetic pieces (crowns, fixed bridges, inlays and onlays).

**TECHNICAL CHARACTERISTICS OF THE PRODUCT:**

Provisory is a cement made of zinc oxide and eugenol for temporary fixation of prosthetic pieces. Provisory forms a thin film, with excellent fluidity and high flow, allowing that the restoration is easily set in place. It presents adequate resistance to support the masticatory loads. It also presents low solubility to the oral fluids and it is of easy removal when necessary.

**COMPOSITION:**

**Base Paste** - Zinc Oxide, Mineral Oil, Vegetable Oil, Barium Sulphate and BHT.

**Accelerator Paste** - Eugenol, Synthetic Resin, Wax e Acetic Acid.

**MODE OF USE:**

1. Dispense equal lengths of the two pastes over a glass plate or a mixture pad. The amount dispensed will depend on the size and type of restoration to be cemented.
2. Dry the dental structures that will receive the cement and keep them isolated under absolute or relative isolation, according to the choice of the professional.
3. Dry the inner surface of the restoration.
4. Mix the two pastes, for approximately 30 seconds, until a thin and uniform paste is obtained.
5. Apply a thin layer of the homogenized cement inside the prosthetic piece.
6. Firmly settle the piece in the right position.
7. Wait for the setting reaction and then remove the excesses, with an adequate instrument.

**Work time:** 30 seconds.

**Set time:** 3 minutes.

**PRECAUTIONS:**

After use, replace the tubes and caps immediately. Keep the tubes well closed, each one with the respective cap. Provisory contains eugenol, which can irritate the skin, oral mucosa or eyes in susceptible people. Professional use.

**WARNINGS AND CONTRAINDICATIONS:**

Do not use in people sensitive or with known allergy to eugenol. In case of allergic reaction to the product, or any other adverse reaction, stop usage.

**EXPIRATION DATE:**

The expiration date is described in the packaging and represents the maximum deadline for use of the product. Do not use after the expiration date. Under adequate storage conditions, Provisory has a shelf life of 36 months.

**STORAGE:**

Keep at room temperature. Keep in original packaging.

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