TRICRESOL FORMALINA

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VALIDEZ:
24 meses.

CONTENT:
Bottle with 10 mL.

INDICATION:
Tricresol-formalin is indicated in cases of necropulpectomy where the pulp canal was not instrumented or it was partially instrumented.

PRODUCT DESCRIPTION:
This solution is formed by a mixture of cresols and formol, being an efficient tissue holder and a powerful antimicrobial agent that has the ability to neutralize toxins in the root canal. Its effect is both by direct contact as by distance through the release of active vapors.

COMPOSITION:
Formaldehyd, O-cresol, ethyl alcohol 96° and purified water.

HOW TO USE:
1. Take a dry and sterile swab of compatible size with the pulp chamber. Then wet it in the solution of Tricresol Formalin.
2. Remove the excess with the aid of sterile gauze and put it in the pulp chamber for subsequent provisional sealing.

PRECAUTIONS:
Product for professional use only. Product contains toxic substances. Keep out of reach of children. Do not use the product after the expiration date. Do not swallow it. If this occurs, seek medical help. Not sterile.

STORAGE AND TRANSPORT:
Store tightly closed in a clean, dry and dark local.

VALIDITY:
24 months.